

Quick Reference Guide

The SonderCare bed was designed with simplicity in mind. Large oversized buttons allow for easy interaction and control of the bed. A clever hand control holder can be easily positioned on either side of the bed. Functional position buttons allow for easy adjustability to common comfort positioning.

Backrest Function
Raise Backrest (S1)

Knee Function
Raise Knee and Foot (S3)

Contour Function
Raise Backrest and Knee simultaneously (S5)

Bed Platform Height
Raise Total Bed Height (S7)

Reverse Trendelenburg
Tilts Bed to Place Head Above Feet (S9)

Comfort Chair
Places Bed in an Upright Chair Position (S11)

Backrest Function
Lower Backrest (S2)

Knee Function
Lower Knee and Foot (S4)

Contour Function
Raise Backrest and Knee simultaneously (S6)

Exit Height
Lowers Bed to Comfortable Exit Height (S8)

Trendelenburg
Tilts Bed to Place Feet Above Head (S10)

Sleep Position
Lowers Bed to Lowest Height for Sleeping (S12)

Lockout Feature

The supplied magnetic chip must be placed in the area marked in red to activate the locking function. At the same time, the respective lock button (see table for symbols for secondary assignment) must be activated. A red LED in the centre of the row of buttons acknowledges that the function lock has been activated.



S2		Locked - NO Functions
S10		Locked - Trendelenburg
S12		Locked - Sleep Position

SonderCare
Safe. Healthy. Living.

1-40-001 Rev 2.0

Quick Reference Guide

The SonderCare bed rail was designed for residents. An all aluminum designed vertical side rail system offers premium quality assistance while allowing for efficient hygienic disinfecting of high touch surfaces. A simple latch mechanism permits the lowering of the rail. Dual stage design promotes increased safety for residents allowing for ergonomic entering and exiting of the bed. 40% coverage (Head Rail) does not require re-positioning to exit bed from a lying position.



Stage 1

Rail is 'free floating' in the Stage 1 position
Simply lift rail to raise to Stage 2

Stage 2

Rail is locked partially while in the Stage 2 position
Simply push Stage 2 lock buttons outward to lower rail back to Stage 1

Stage 3

Rail is locked while in the Stage 3 position
Simply push Stage 3 lock buttons inward to lower to Stage 2

SonderCare
Safe. Healthy. Living.

1-40-001 Rev 2.0